PL - Separator palców stóp

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EN - Toe Separator

How to use: Wash and thoroughly dry your feet before putting on the separators. Gently stretch the separator and place it between your toes, starting from the base of the toes. Initially, use the separators for 10–15 minutes per day, gradually increasing the wearing time as your feet adjust. They can be worn both barefoot and with loose-fitting footwear. After removing the separators, wash them in warm soapy water and allow them to air dry.

Cleaning and care: Hand wash the separators in warm water with mild soap. Avoid using harsh detergents or chemicals that may damage the material. After washing, rinse thoroughly and allow to dry naturally. Store in a cool, dry place away from direct sunlight.

WARNING: Do not use while sleeping, near a heat source, or during exercise. Do not apply additional pressure to the product. Always ensure proper air circulation around the foot. Regularly check the condition of the separators; if you notice any damage or excessive wear, replace them with new ones. If skin discoloration occurs, discontinue use immediately. Not suitable for children, elderly individuals, or those who cannot remove the product.

independently. Do not use on irritated or damaged skin, wounds, or blisters. Do not use in combination with creams applied to the skin or if you have rheumatoid arthritis, poor circulation, nerve damage, sensory disorders, any skin disease, Raynaud's disease, diabetes, or if you are pregnant. For external use only. Do not use if you are allergic to latex. Do not use for more than 2 hours at a time. This product is not intended to prevent injuries; any discomfort may indicate an underlying issue, and a visit to a primary care physician is recommended. This product is not a substitute for professional medical care. Plastic bags can be hazardous. To avoid the risk of suffocation, keep them out of reach of children and infants.

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