PL - Aparat na haluksy

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EN - Bunion Corrector

Instructions for Use: Before putting on the corrector, wash and dry your feet. Gently place your toes into the appropriate compartments and fasten the bunion corrector. The corrector should be comfortable and should not exert excessive pressure on your feet. If necessary, loosen the fastening. Initially, it is recommended to wear the bunion corrector for a short period (e.g., one hour per day) to allow your feet to adjust. Gradually increase the wearing time. After each use, wash the corrector thoroughly with soap and water and dry it.

Warnings: Read the user manual before use and keep it for future reference. Avoid using on cracked, irritated, or damaged skin. If you experience pain, irritation, or discomfort, discontinue use immediately. Continuous use for long periods is not recommended. The bunion corrector is best used at night or while resting. Perform foot exercises before and after wearing the bunion corrector. Consult a doctor before use if you have diabetes, circulation disorders, or foot conditions. Monitor your feet and body reactions while using the corrector. If you notice any concerning symptoms, discontinue use and consult a doctor. Keep the product clean. Wash in lukewarm water with a mild detergent. Do not dry the bunion corrector directly near heat sources. This product is not a toy. Keep out of reach of children.

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