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EN - Folding cane with height adjustment

Instructions for Use: To assemble the cane, insert each lower segment into the segment above it, and then check the connection of each part. To disassemble, pull each lower segment out from the segment above and fold the parts together. Thanks to the flexible cord inside the cane, all segments remain in the correct order. To adjust the height of the cane, press the metal pin located at the top and slide it up or down until the desired height is achieved. Before each use (by pressing on the handle), make sure the locking pin is securely in the hole on the cane shaft. The height of the cane can be adjusted in 2.5 cm increments.

WARNING: Before each use, ensure the locking pin is firmly seated in the hole on the cane shaft. If unsure about the correct height, consult a doctor. Exercise caution when using the cane on wet floors, tiles, and other slippery surfaces, as well as on uneven terrain. Do not exceed the maximum user weight of 100 kg. Do not use if the rubber tip is damaged or worn. Replace the rubber tip immediately if worn out or destroyed. If the cane is damaged, do not attempt to repair it yourself – discontinue use and have it repaired or replaced. The manufacturer is not responsible for any injury or damage resulting from improper use of this product.

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