PL - Stabilizator łokcia

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English

Elbow Stabilizer Warnings and Precautions (In accordance with GPSR)

CAUTION! This product is designed to support and stiffen the elbow joint and for injury prevention. In case of doubt or persistent pain, always consult a doctor or physical therapist.

- **Medical Consultation:** Before beginning use for therapeutic purposes, it is recommended to seek medical advice.
- **Do Not Apply to Open Wounds:** The stabilizer should not be placed directly on damaged skin, open wounds, abrasions, or in case of visible swelling or bruises that may require direct medical attention.
- **Avoid Excessive Pressure:** The stabilizer should fit snugly but must not cause excessive pressure, tingling, numbness, or blue discoloration of the hand. Too much pressure can impede blood circulation. If you notice these symptoms, immediately loosen or remove the strap.
- **Allergic Reactions:** In case of allergic reactions (e.g., itching, rash) to the material (neoprene), discontinue use immediately and consult a doctor.
- **Increase in Pain:** If you experience increased pain or new discomfort while using the stabilizer, remove the strap and consult a specialist.
- **Storage:** Store the product in a dry and cool place, away from direct sunlight and heat sources, to preserve the properties of the material (neoprene).
- **Intended Use:** The product is intended only for external use on the elbow joint.
- **Universal Use:** The stabilizer is universal – it fits both the left and right elbow and is suitable for women and men.

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